


BackLit Tray Setup

Disassembling and assembling the BackLit Tray

1. Unplug BLT from wall outlet; unplug T-5 cord from T-5 receptacle on light source.
2. Lift the upper tray **A** out of the lower tray **B**.
2. Remove the white light diffusion plate **C** (Tip: a pen can be inserted into the semi-circular notch of the diffusion plate to assist removal.), revealing the light source lower compartment **D**.
3. Slide out the light source **E**.

Reverse the steps to assemble the BLT-5 corded model.

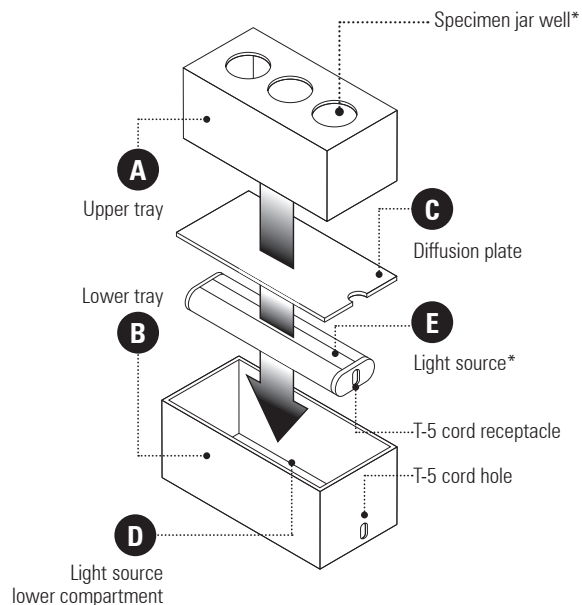

* Wells in upper tray and light source will vary

BackLit Tray Operation

BLT-5: Turn ON/OFF with in-line toggle switch.

BackLit Tray Cleaning

How do I clean BLT surfaces?

Alcohol or hydrogen peroxide wipes can be used to quickly and easily clean the BLT outer surfaces as needed between cases.

The BLT upper tray, lower tray and diffusion plate can be rinsed under water with gentle detergent, and hand dried.

Can I use bleach to clean the BLT?

Bleach can be used, but can leave a hazy residue, which can be easily wiped clean with alcohol, hydrogen peroxide or ammonia based cleaners.

Can I submerge the light source? Is it waterproof?

No. Do not run it under water or submerge it. Use bleach or alcohol wipes to clean.

Can I use heat to dry the BLT parts?

No. Do not expose any parts to heat or heated drying, as this can melt and deform the unit.

For more information visit:

<https://www.specivue.com/faq/>